Index

Deutsch:

Ortho Dynofix Fixateur Extern	DYNOFIX 1-16
Dynamic Fixateur Extern	DYNAMIC 1-12
Hand / Finger und Mandibula Fixateur Extern	H-F.FIX 1-24

Englisch:

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External Fixateur ORTHO DYNO FIX

8-0414
Pilon Clamp with long and standard fixator

Cat. No.:	Type	Lenght	Weight gr
8-0410	Long	280	700
8-0412	Standard	250	640

Limb Lengthener Fixateur
Femur and Tibia
DINO-FIX

Cat. No.:	Type	Lenght mm	Weight gr	Lengtening
8-0430	Long	230	540	80
8-0432	Standard	200	480	60
8-0434	Short	180	420	50

LIMB Reconstruction Fixateur for Femur and Tibia **DINO-Fix**

8-0440
Short Model
200 mm long with 2 clamps

8-0442
Standard Model
250 mm long with 3 clamps

8-0444
Long Model
300 mm long with 3 clamps

Cat. No.:	Type	Lenght mm	Weight gr
8-0440	Short	200	410
8-0442	Standard	250	480
8-0444	Long	300	560

ORTHO DYNOFIX EXTERNAL FIXATEURS MODUL SYSTEM

External Fixateur for Pelvic

Pelvic External Fixator with Clamp	
Cat. No.:	Model
8-0450	Short
8-0452	Standard
8-0454	Long

Rod and Clamp Fixateur Multipurpose

Rod and Clamp Fixator Multipurpose	
Cat. No.:	Model
8-0460	Short
8-0462	Standard
8-0464	Long
8-0466	Small

Axial Fixator for Humerus and Small Tibia

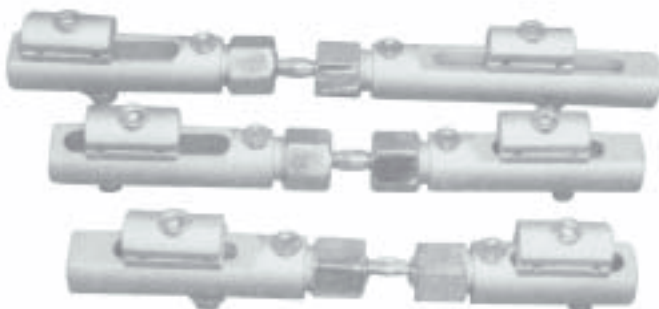
Material: Special Aluminium Alloys
Type: Dynamic Axial, Long, Standard
Move Angle: 45°
Diameter: 25 mm

8-0474
T Clamp

Cat. No.	Type	Length mm	Lengthening mm	Weight gr
8-0470	Standard	170	25	300
8-0472	Long	195	46	340

External Fixator for Hand - Wrist and Radius


Cat. No.	Type
8-0480	Short
8-0482	Standard
8-0484	Long

ORTHO DYNOFIX EXTERNAL FIXATEURS MODUL SYSTEM

Cat. No.:	
9-023	Drill Bit 3.5 mm extra long 180 mm
9-027	Drill Bit 4.5 mm extra long 180 mm
1-1864	Drill Bit 2.7 mm 150 mm
1-1866	Drill Bit 4.0 mm 150 mm
1-1867	Drill Bit 4.8 mm 150 mm

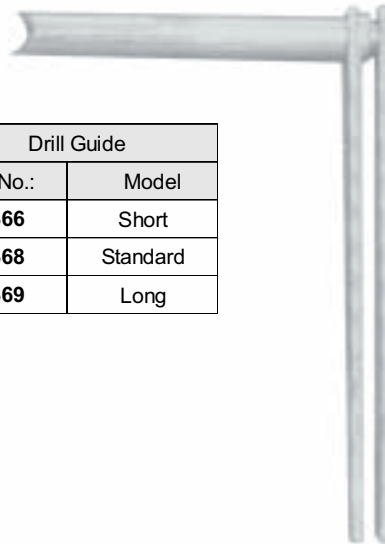
Drill Guide	
Cat. No.:	Model
8-0366	Short
8-0368	Standard
8-0369	Long


Schanz Screws (with Trocar Point and Triangular Shaft)

Order No.	mm \varnothing	Length mm	Thread length mm
8-1200	3.00	60	20
8-1202	3.00	80	20
8-1220	4.00	60	25
8-1222	4.00	100	25
8-1224	4.00	130	25
8-1228	4.00	170	25
8-1240	4.50	100	50
8-1242	4.50	120	50
8-1244	4.50	150	50
8-1246	4.50	170	50
8-1248	4.50	200	50
8-1250	4.50	250	50
8-1260	5.00	100	50
8-1264	5.00	125	50
8-1266	5.00	150	50
8-1268	5.00	170	50
8-1270	5.00	200	50
8-1272	5.00	250	50
8-1280	6.00	100	30
8-1284	6.00	150	50
8-1285	6.00	170	50
8-1286	6.00	200	50
8-1287	6.00	250	50

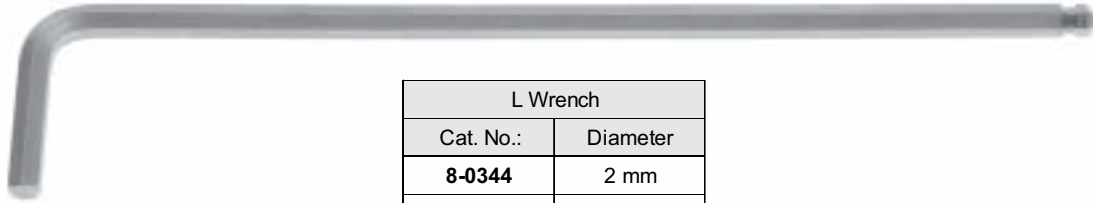L Wrench	
Cat. No.:	Diameter
8-0344	2 mm
8-0343	3 mm
8-0339	4 mm
8-0338	5 mm
8-0345	6 mm

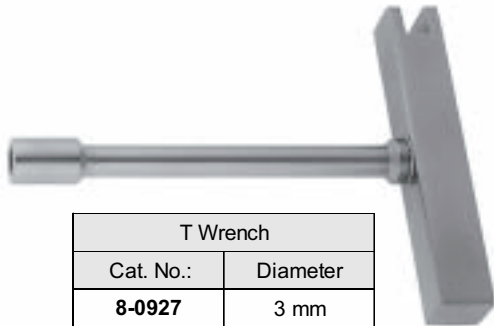T Wrench	
Cat. No.:	Diameter
8-0927	3 mm
8-0928	4 mm
8-0929	5 mm
8-0930	6 mm

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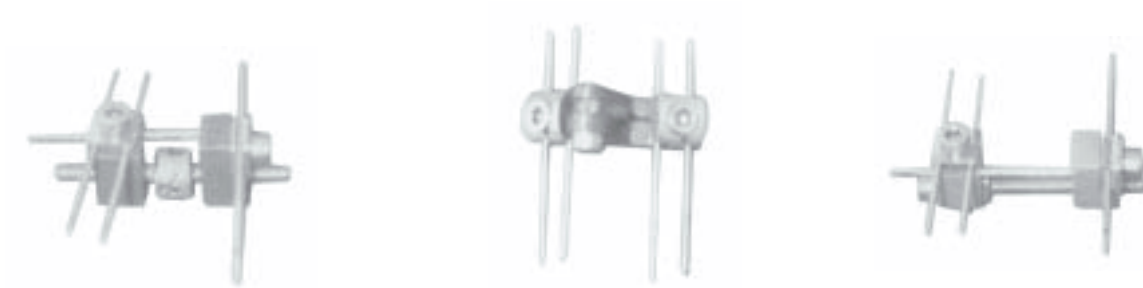
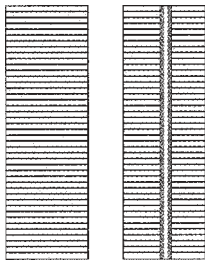
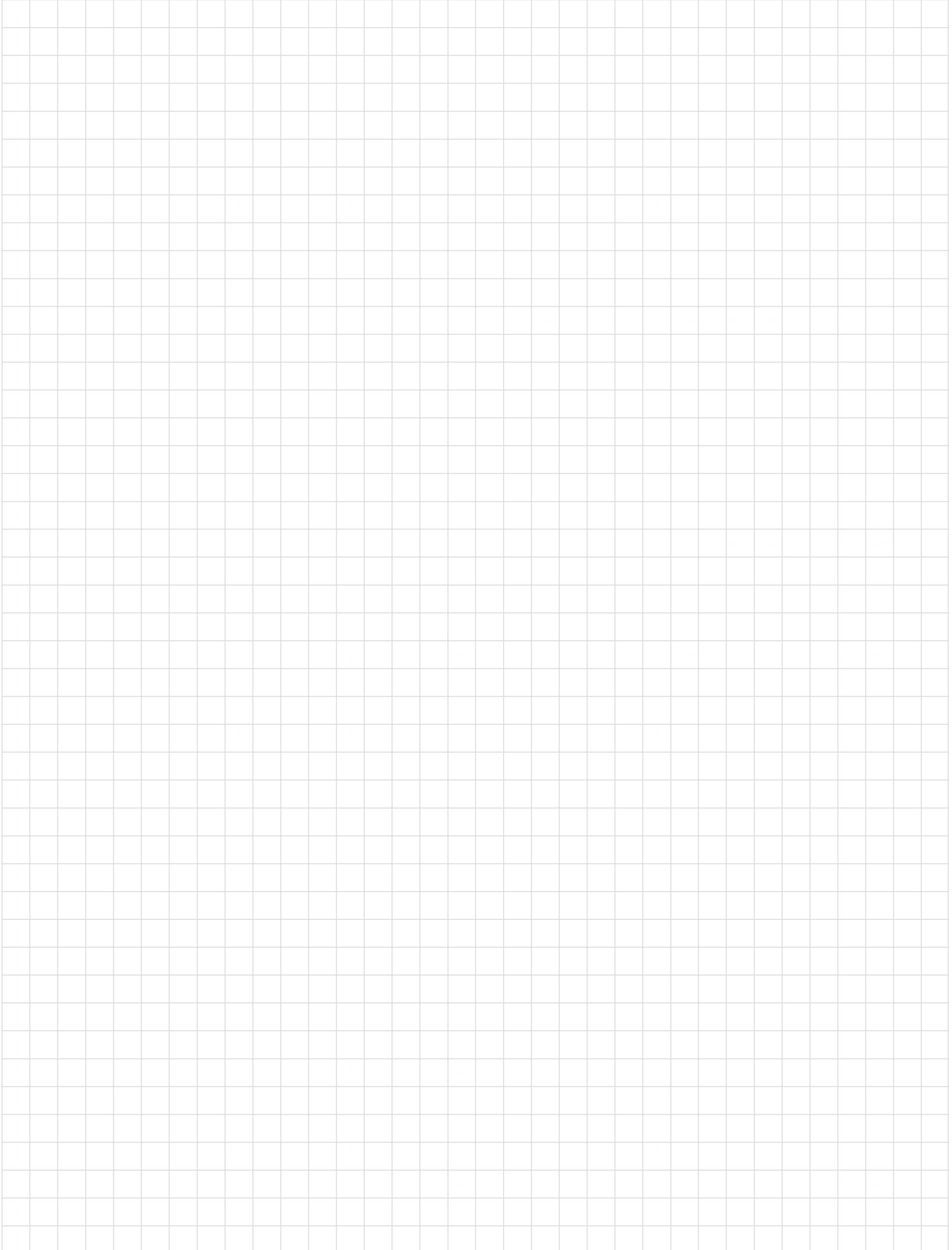
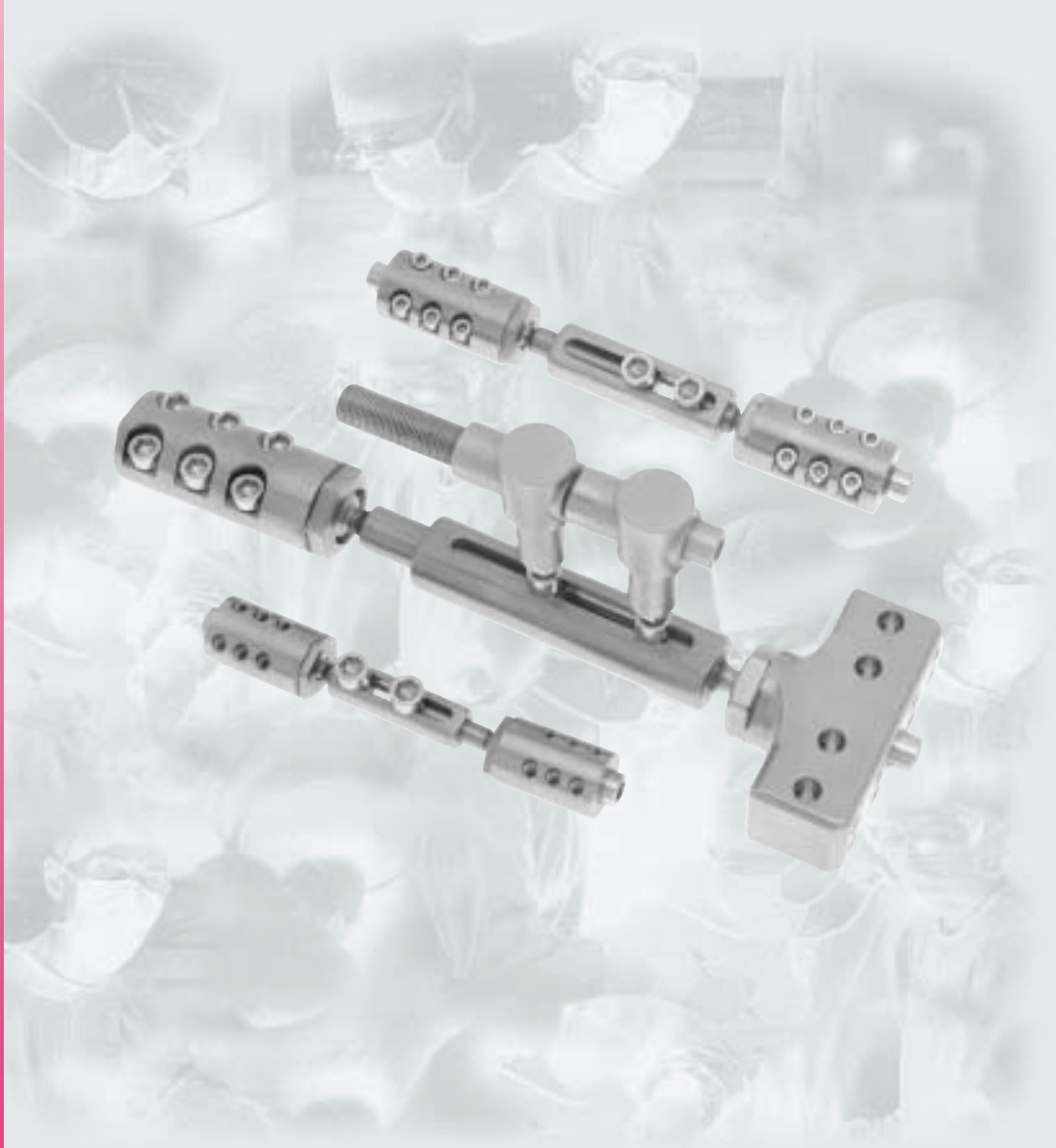
ALL TYPES MADE OF TITANIUM

1. extremely light weight
2. high stability under load

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**Self - cutting with
drill bit and spiral
groove.**

**Advantage:
Pre - drilling is not
necessary.**


Cat. No.	mm \varnothing	Length mm	Thread length mm
8-1300	3.00	60	20
8-1302	3.00	80	20
8-1310	5.00	150	50
8-1312	5.00	170	50
8-1314	5.00	200	50
8-1320	6.00	150	50
8-1322	6.00	170	50
8-1324	6.00	200	50

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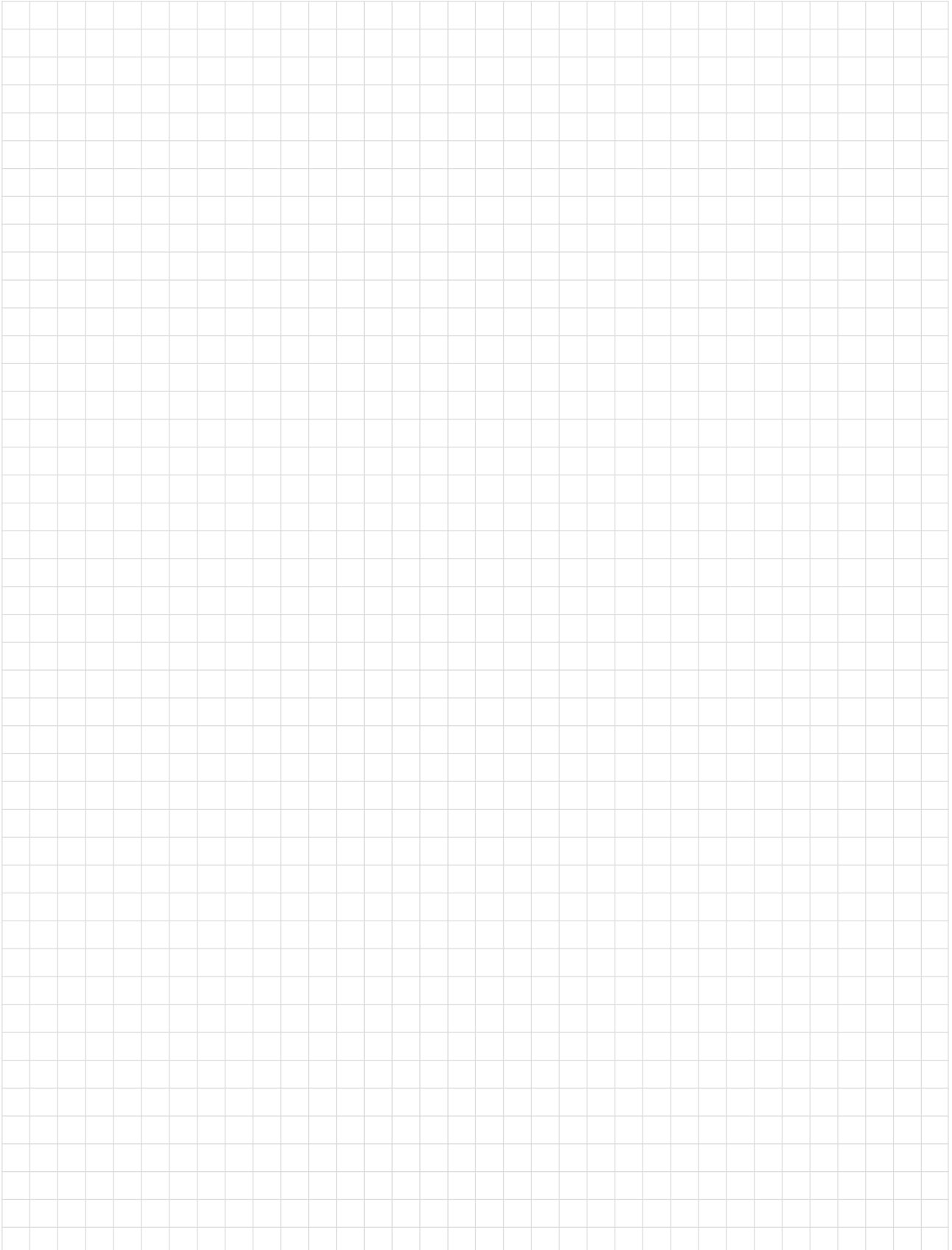Picture 1

Fixateur Externe serves as well as drill template. To obtain the requested distraction or compression the Fixateur Externe is adjusted to medium length in straight position. Introduce drill sleeve into distal hole and fix slightly with screws.

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Picture 2

Place Fixateur Externe with both drill sleeves on bone. Introduce self - cutting Schanz screws through drill sleeves (no additional drilling requested). Introduce and fix further Schanz screws in the same way.

Caution: When using Schanz screws ref. 8-1200 to 8-1272 pre - drilling is necessary.

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
Picture 3

Final fracture reposition is achieved under X - Ray control by loosening the ball joints. Fix tightly ball joints and screws.

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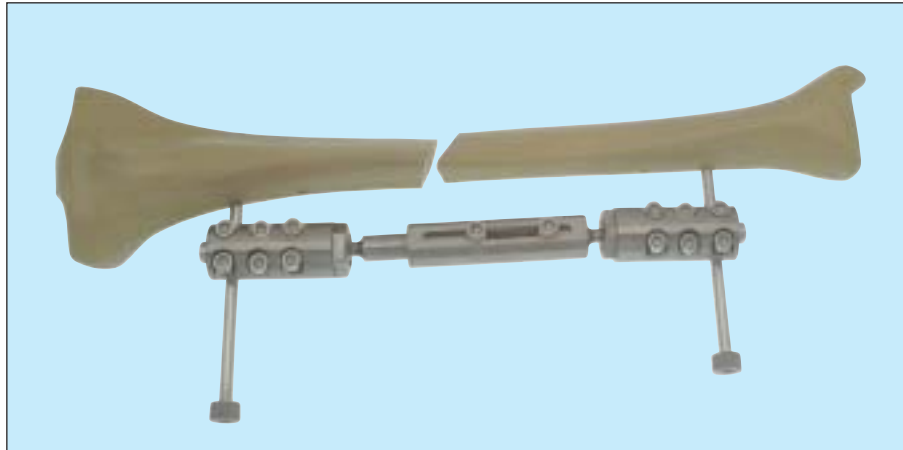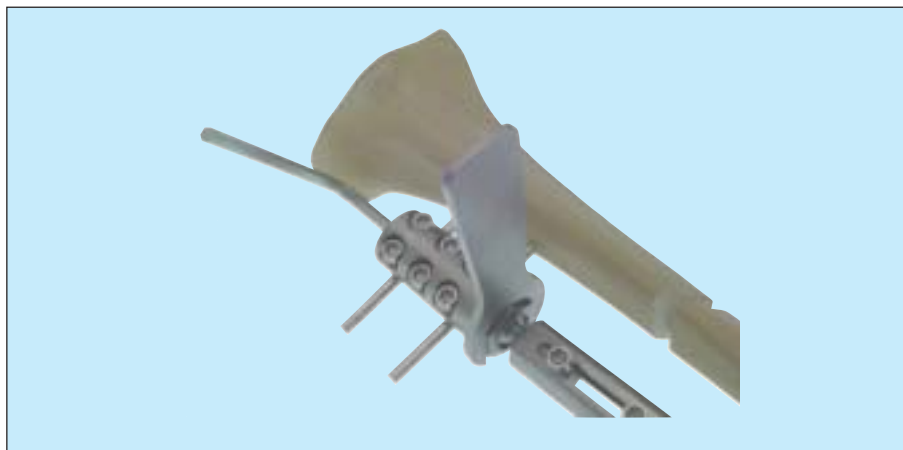
HAND / FINGER AND MANDIBULA FIXATOR HAND / FINGER UND MANDIBULA FIXATOR

erich - FIX SURGICAL TECHNIQUE AND PRODUCT DESCRIPTION

Introduction

Severe hand injuries are demanding for both the surgeon and the patient. Here the primary goal of treatment is to restore as much function as possible in the shortest possible time. The Hand Fixator "**erich - FIX**" offers an unusually versatile and economic system for the treatment of complex, as well as compound fractures.

Description

The handfixator "**erich - FIX**" consist of two or more titanium blocks for the fixation of the threaded pins. The gliding blocks are adjusted to the desired length on a threaded rod. **erich - FIX I** and **II** also permit secondary distraction or compression. Joint motion can be excluded permitted. **erich - FIX III** is slim enough to interdigital application, whereby tissue protection is guaranteed by its smooth design.

Preoperative Planing

For optimal fixation careful preoperative planning is essential. Hereb injuries to the nerves and the vessels of the fingers are be avoided. Where joints are be mobilized lateral positioning of the fixator is necessary. If bone grafting is anticipated, all necessary measures should be implemented in advance. Patient must be onformed of treatment demands and potential complications.

An image intensifier is usually necessary to assure correct application. Certain fracture patterns require additional fixation with screws, a tension band or a K wire.

Indications (Examples)

- Fractures of the shaft, simple or comminuted
- intraarticular fractures
- compound fractures
- trauma or tumor - related bone defects of the carpus, metacarpus or phalanges
- arthrodesis
- distraction osteogenesis (i. e. lengthening of the thumb)
- metacarpal segment transport
- callotaxis lengthening of the capitate after resection of the lunate bone (for treatment of Kienböck's disease, stage III)
- fractures of the mandibula

Fig. 1: Description of the fixator

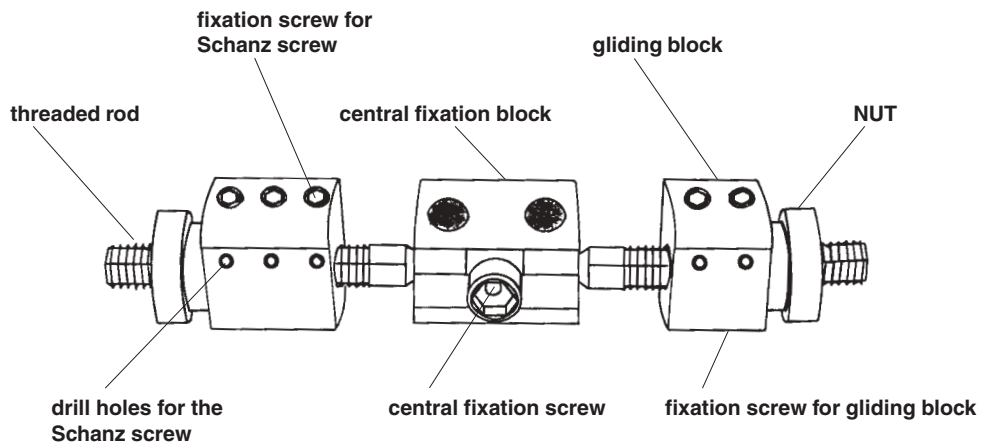
Fig. 2: Diagramm of fixator assembly


Fig. 3: Holding lever

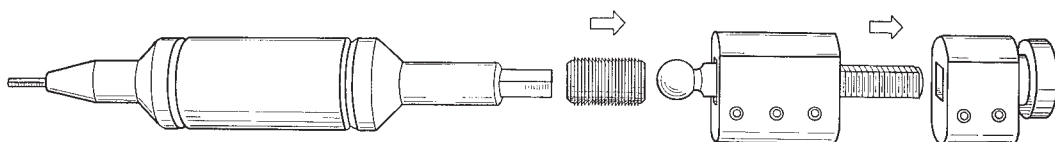The fixation screws are secured by means of holding lever

- wide opening lever for **erbrich - FIX I and II.**
- small opening lever for **erbrich - FIX III.**

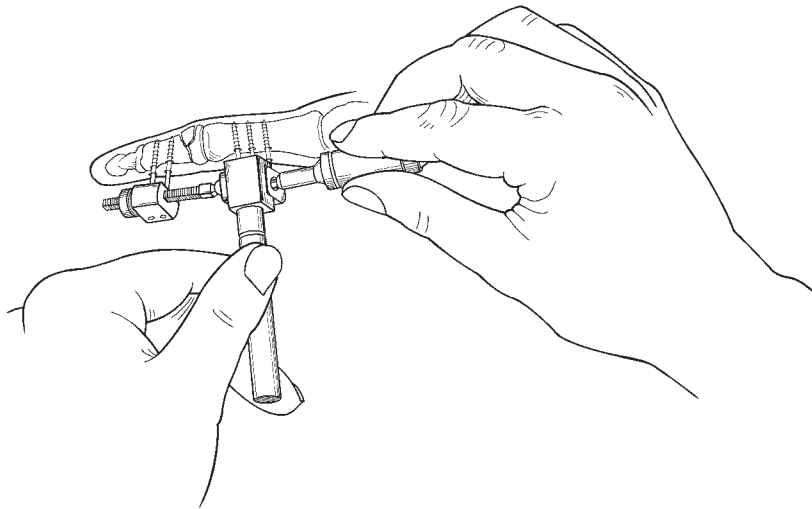
Fig. 4: D 3 : Fracture of the base of the middle phalanx

dorso - lateral application of **erbrich - FIX III.**

**D 4: Comminuted fracture of proximal phalanx: dorso - lateral transarticular
erbrich - FIX III**

The threaded pins should be placed beside the proximal extensor tendon.

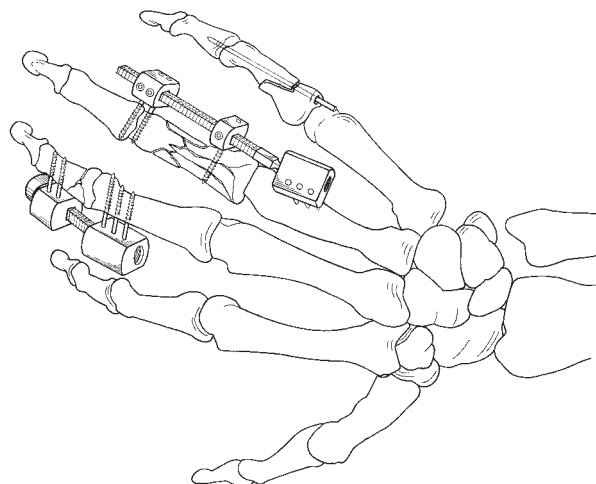
Fig. 5: Comminuted intraarticular fracture of the head of the proximal phalanx

interdigital transarticular application of **erich - FIX III**.

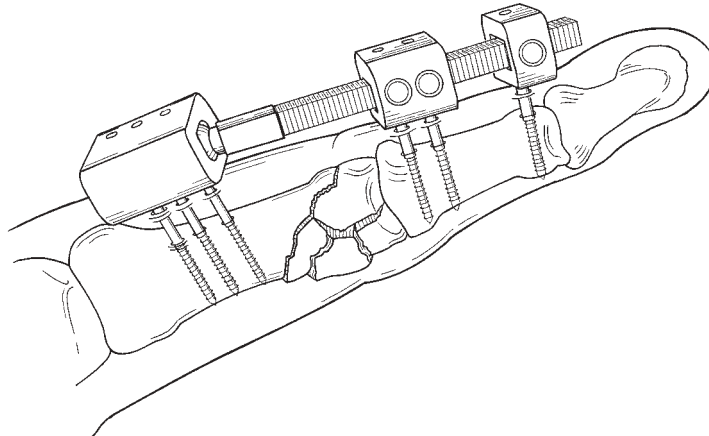
Fig. 6: Defect of the IIIrd metacarpal Bone

Cancellous bone graft, compression osteosynthesis with a dorsally applied **erich - FIX II**.

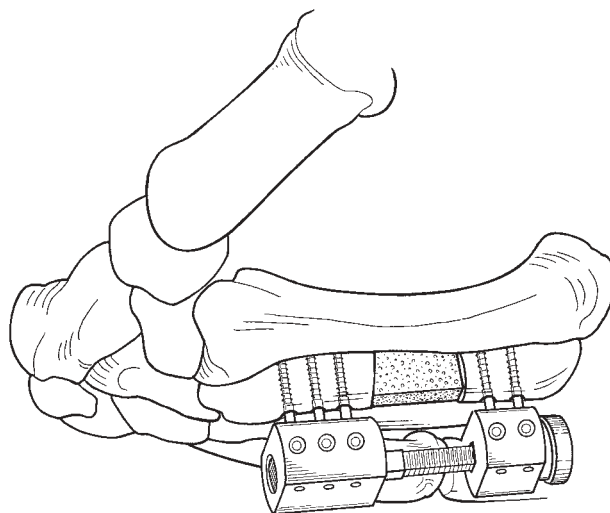
Fig. 7: Arthrodesis

Compression arthrodesis in the PIP - joint using **erbrich - FIX II**.

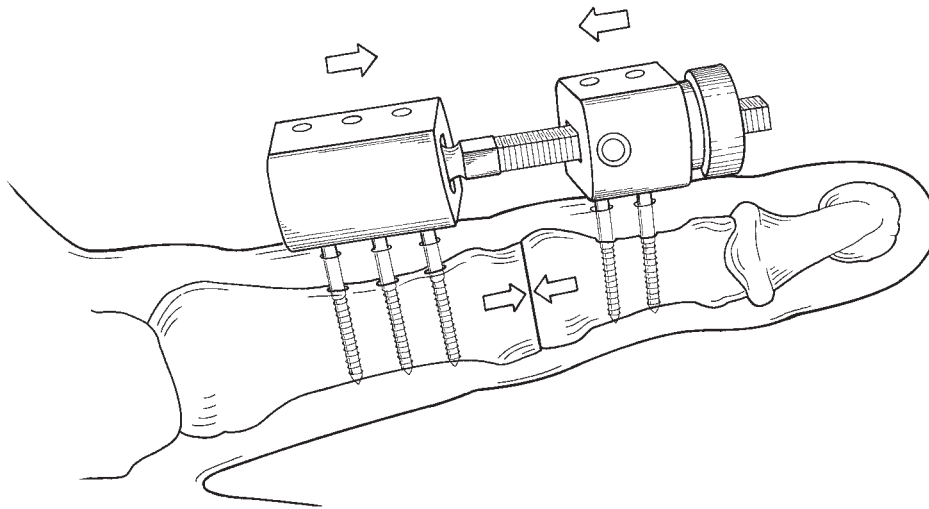
Fig. 8: Distraction osteogenesis following amputation of the thumb

Osteotomy of the first metacarpus.
Bilateral application of **erbrich - FIX II**. Distraction should begin on the 10th day (1 mm / day)


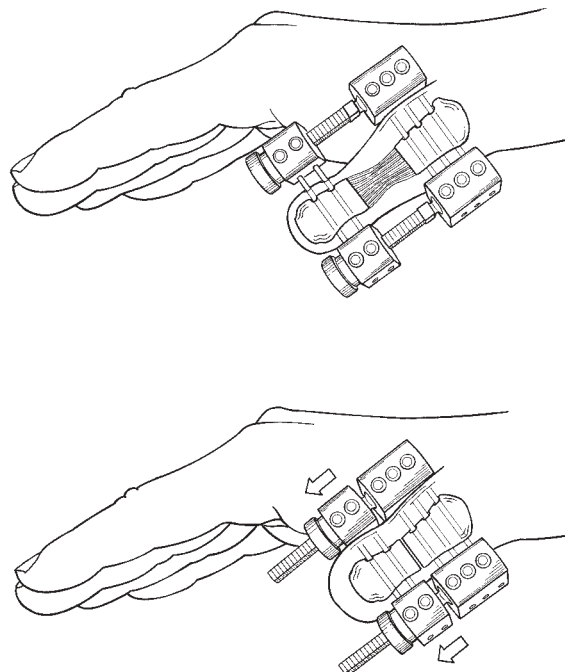
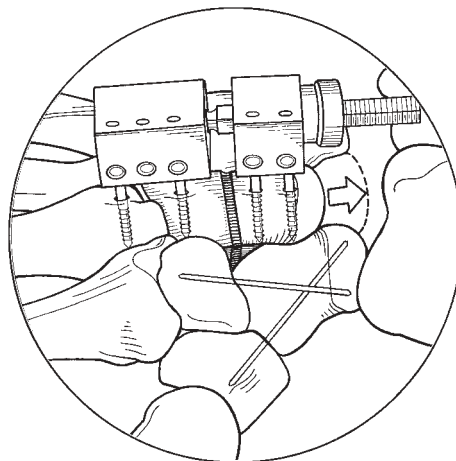
Fig. 9: Distraction of capitate

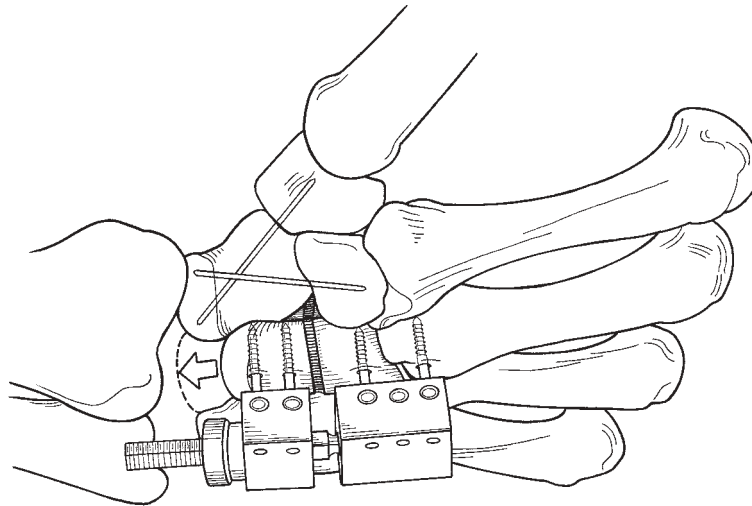Following resection of the lunate due malacia stage III, osteotomy of the capitate is performed together with temporary fixation of the scaphlo - capitate and scaphulo - trapezio - trapezoid joint. Distraction should begin on the 10th (1 mm / day).

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Handchirurgie, Mikrochirurgie, Plastische Chirurgie 29 (1997) 10 - 19

Fig. 10: Active or passive motion.

Lateral positioning and central loosening of the joint fixation screws permit active or passive motion therapy. The axis of motion of the fixator should be parallel to the joint axis. The degree of healing should be suited for mobilization.


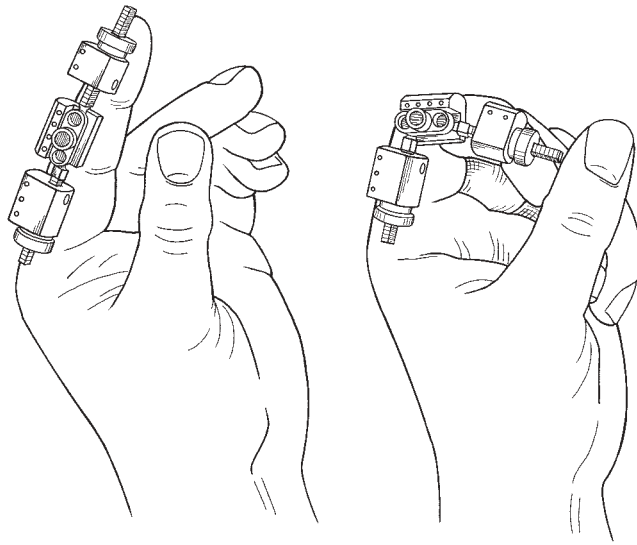
Abb. 11: Fracture of the mandibula

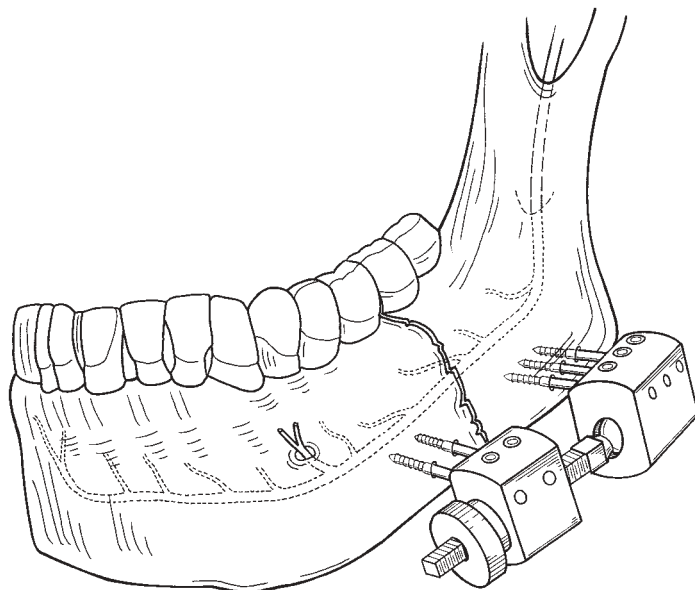Application of **erbrich - FIX II** at the lower mandibular rim. Preservation of the mandibular nerve is essential.

HAND / FINGER AND MANDIBULA FIXATOR HAND / FINGER UND MANDIBULA FIXATOR

In order to cut K - wires place pin - block temporarily directly against the skin. After cutting the K - Wires push the pin - block back that the ends of the pins are covered. There should be enough space between skin and fixator to avoid maceration.

Postoperative Care

The majority of injuries can be stabilized in an ambulatory setting. Postoperatively wounds are covered by a light dressing. After three days dressings are changed or removed. The entrance points of the pins should be cleaned with soap water. Pin tract infections may be initiated by frequent manipulation of the entrance points.

The frequency of X - ray controls should be based on the dislocation risk of the stabilized fracture.

Pin tract infections require oral antibiotics and the interruption of active or passive motion. Occasionally widening of the entrance points by incisions is advisable. Apparent loosening requires pin removal. Reinsertion is seldom necessary as dislocation usually occurs at the end of the fixation period. Primary consolidation takes place within 3 - 6 weeks, depending upon the involved bone type and the extent of soft tissue trauma. The **erbrich - FIX** can usually be removed without local anaesthetics. Further physiotherapy is needed to optimize functional outcome.

HAND / FINGER AND MANDIBULA FIXATOR HAND / FINGER UND MANDIBULA FIXATOR

Standard

T 250 Hand / Finger und Mandibula Fixator Set

Cat. Nr.	Beschreibung	Stück
8-2506	erbrich - FIX Nr. I	1
8-2510	erbrich - FIX Nr. II	1
8-2520	erbrich - FIX Nr. III	1
8-2522	Teflon Steri Etui	1
8-2530	Ersatzspindeln 40 mm	1
8-2532	Ersatzspindeln 50 mm	1
8-2534	Ersatzspindeln 60 mm	1
8-2536	Ersatzfeststellschrauben	20
8-0342	Inbusschlüssel 1,5 mm	1
8-0339	Inbusschlüssel 4,0 mm	1
8-2540	Schraubendreher	1
8-2550	Haltebolzen schmal	1
8-2552	Haltebolzen breit	1
7-300	Kirschnerdrähte ø 1,8 mm 70 mm lang, 30 mm Gewinde	12
7-301	Kirschnerdrähte ø 1,6 mm 70 mm lang, 30 mm Gewinde	12
zusätzlich empfohlene Instrumente:		
8-2560	Haltezange	1
7-430	Zwickzange	1
7-432	Zwickzange	1
7-388	Kirschner Extraktionszange 130 mm	1
7-390	Kirschner Extraktionszange 180 mm	1
7-400	Ausziehzange Chrom Vanadium	1
7-402	Ausziehzange rostfrei	1

Standard


T 250 Hand / Finger and Mandibula Fixator Set

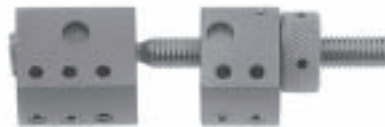Cat. No.	Description	Pieces
8-2506	erbrich - FIX No. I	1
8-2510	erbrich - FIX No. II	1
8-2520	erbrich - FIX No. III	1
8-2522	Teflon Steri Box	1
8-2530	Threaded Rod 40 mm	1
8-2532	Threaded Rod 50 mm	1
8-2534	Threaded Rod 60 mm	1
8-2536	Fixation Screws	20
8-0342	Allen Key 1.5 mm	1
8-0339	Allen Key 4.0 mm	1
8-2540	Screw Driver	1
8-2550	Holding Lever small	1
8-2552	Holding Lever wide	1
7-300	Kirschner Wire ø 1.8 mm 70 mm long, thread 30 mm	12
7-301	Kirschner Wire ø 1.6 mm 70 mm long, thread 30 mm	12
Recommended Instruments		
8-2560	Holding Forceps	1
7-430	Wire Cutter	1
7-432	Wire Cutter	1
7-388	Kirschner Wire Extracting Pliers 130 mm	1
7-390	Kirschner Wire Extracting Pliers 180 mm	1
7-400	Flat Nose Parallel Pliers Chrome Vanadium	1
7-402	Flat Nose Parallel Pliers stainless steel	1

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erbrich - instrumente GmbH